


Selecting and Downloading Your Templates

Once you have built a selection of templates in the My Templates screen it is possible to download all of these templates in one go, this makes it easy to update your entire template library in only a few clicks.

On the 'My Templates' screen you will see a 'Select All' button, it is important to note that this only selects all the templates currently displayed on the screen. Before you click 'Select All' you should use the filter buttons at the top of the page to show the templates required, in particular you should change the 'Show Me' filter to "Everything". If you have a large number of templates it may be more efficient to use the 'Manufacturer' filter to download templates in smaller sets, the website will automatically prevent you from downloading more than 150 templates at one time.

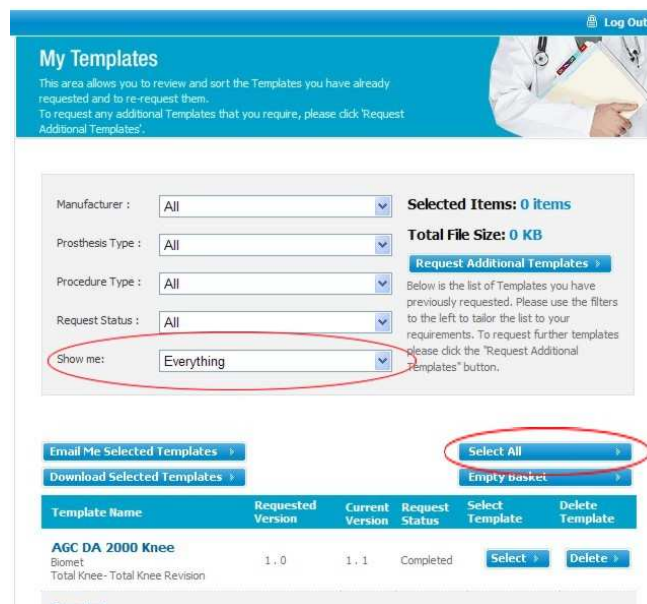

How to Install Your Templates

Once you have received the ZIP file containing the templates they will need to be correctly installed to make them available in OrthoView, to do this please follow the steps below:

Extract the contents of the ZIP file to an empty folder somewhere on a workstation that has OrthoView installed. If you have a server based installation of OrthoView the templates only need to be installed once and they will automatically be available to all users. If you have individual workstations the templates will need to be installed on each workstation.

If you know the location of the folder that will contain the OrthoView templates then you can copy the .ovt files directly in to it. If you do not know the location then you can start an automatic installation routine by double-clicking anyone of the .ovt files and following the onscreen prompts.

If you have web browser based installation of OrthoView then instead of double-clicking you will need to enter a URL in to your web browser to start the installation process, the URL will be provided as part of the installation documentation.